# Parent / Guardian Caution

Instructions for Safeguarding Your Child's New Crowns

Dear Parent or Guardian,

With your consent, one or more of your child's teeth have been restored with NuSmile® Pediatric Crowns. CAUTION!! Crowns can be susceptible to becoming dislodged or to fracture when subjected to extreme conditions and certain types of habits including eating or chewing on hard objects such as toys or hard candy; and/or in the case of traumatic injury.

If your child has a habit of chewing on hard objects such as toys, car keys or anything which might place unusual stress on the teeth, please discontinue this habit. Please avoid giving your child sticky or very hard foods such as candy, hard bread or ice as these may dislodge or cause damage to the crown. Safeguard as possible against any trauma or hard blows to the teeth.

I understand that neither this office nor the manufacturer extends a warranty against breakage of the cosmetic facings or ceramic structure of these crowns. However, in the event of the crown becoming dislodged or fractured within \_\_\_\_\_ ( ) days of placement, our office will replace or repair the cosmetic crown(s) at no charge. After \_\_\_\_\_ ( ) days, there will be a replacement or repair charge which will not be covered by insurance.

_____	_____/_____/_____ Date of Birth
Child's Name	Date of Birth
<input checked="" type="checkbox"/> _____	_____
Parent / Guardian Signature	Date
<input checked="" type="checkbox"/> _____	_____
Dentist or Staff Member	Date

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